

Ambu[®] BlueSensor NEO X




USER GUIDE

Ambu


INDICATIONS FOR USE

The unique no-touch application system makes it easy and quick to apply the electrode with gloves. The four-leaf clover shape and flexible, breathable backing ensure reliable performance throughout long-term ECG monitoring.


PLACEMENT

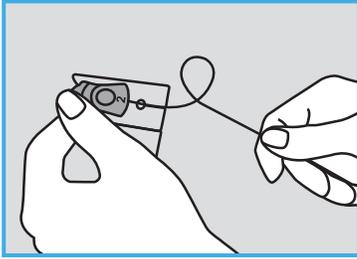


Straighten the lead wire.

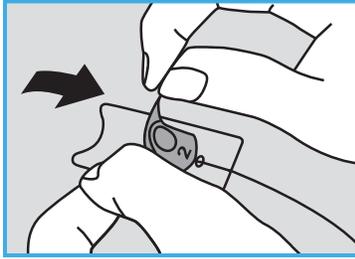


Remove the electrode from the plastic liner by pulling it towards you - using the white tab "1".

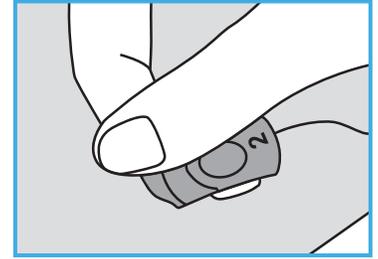


Place the electrode on the patient's skin.

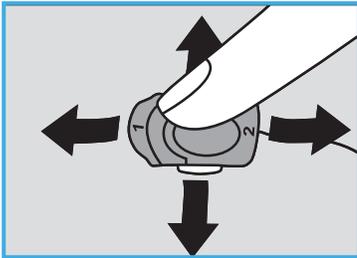

Gently apply pressure with your finger on each of the four leaves around the edges of the electrode.

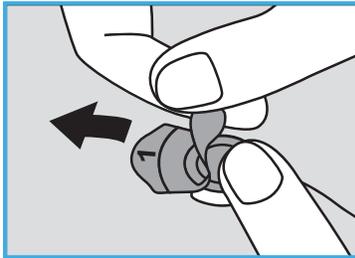


Remove the tab "2" by pulling it upwards, parallel to the skin. At the same time, hold the electrode in place with your other hand.

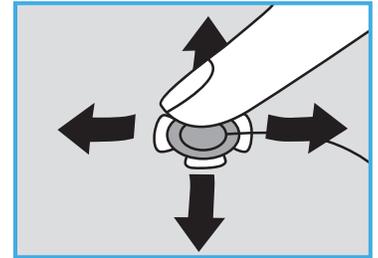


Gently apply pressure with your finger on each of the four leaves around the edges of the electrode.


REMOVAL

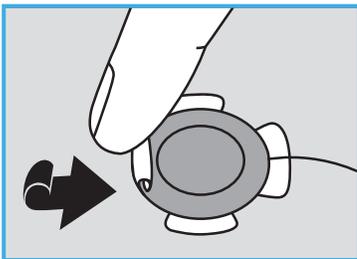

Gently roll the leaves of the electrode towards the center.

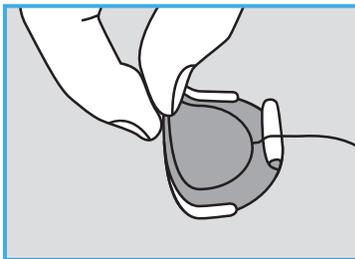

Lift up the electrode with your finger between the adhesive and the patient's skin.

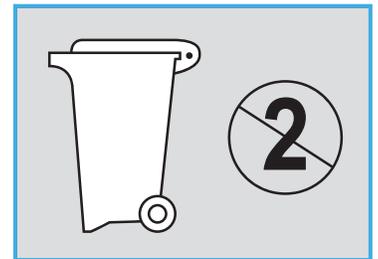

Dispose the electrodes - the NEO X electrodes are not repositionable.