



User Manual: Ives Electrode Harness

J.R. Ives, December 22, 2017

Ives EEG Solutions; 25 Storey Ave., #118, Newburyport, MA, 01950; ph: 978-358-8006; fax: 978-358-7825; www.iveseegsolutions.com

Cleaning	Use warm water with a soft brush and detergent to remove all foreign material. Dry the electrode harness with paper towel and store them by hanging in the air. Do not store the harness in a plastic bag.
Disinfection	Use a bleach wipe or Sani-Cloth disposable wipe to disinfect the harness. Do not soak the harness in a disinfection solution. DO NOT SOAK (even in water) the electrode harness. Again, dry electrode harness with paper towel and hang in air to dry for storage.
Sterilization	If sterilization is required by your laboratory's protocol, you will need to follow procedures and recommendations of your institution.
Storage between use	Hang electrode harness in air. Do not store it in a sealed bag.
