

Ambu[®] Regulator

How to Set the Pain Pump Flow Rate




Remove blue lock guard.



Gently pull white dial up on regulator.



To set the Regulator to desired flow rate, rotate the white dial and align with black position indicator. Increase flow rate turn counter clockwise. Decrease flow rate turn clockwise.

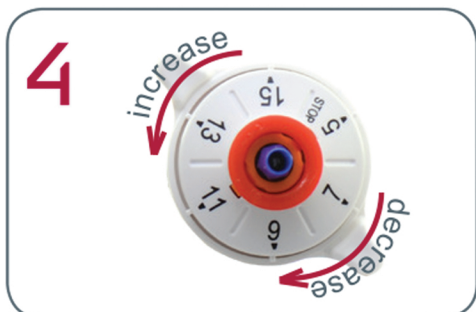

Caution: Regulator is equipped with positive stop at max and min flow rates. Do not force past the positive stop, as this may damage the regulator.




Push the white dial back in place to set flow rate.
Avoid pressing the red ring at all times.



To secure flow rate **temporarily**, insert blue lock guard between the red ring and the white dial.



Please DO NOT Touch the RED Locking Ring

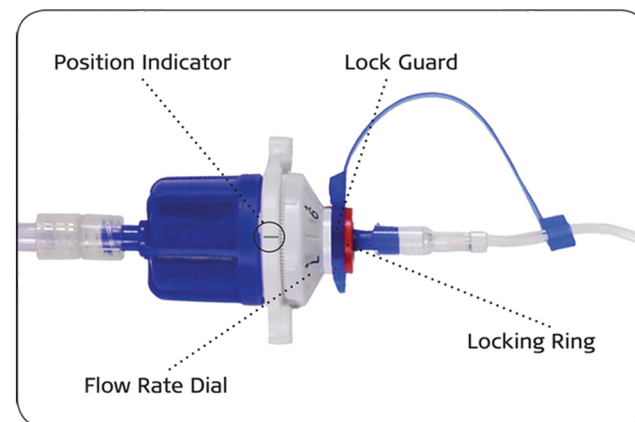

For Questions regarding the Ambu ACTION[™] Pain Pump please call our 24 hour clinical Hotline: (800) 395-2890.

Ambu[®] Regulator

Locking the Regulator




Remove blue lock guard.



Gently pull white dial up on regulator.



To set the Regulator to desired flow rate, rotate the white dial and align with black position indicator. Increase flow rate turn counter clockwise. Decrease flow rate turn clockwise.

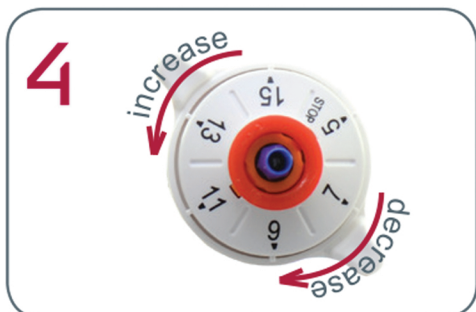

Caution: Regulator is equipped with positive stop at max and min flow rates. Do not force past the positive stop, as this may damage the regulator.




Push the white dial back in place to set flow rate.
Avoid pressing the red ring at all times.



Ensure both C-Clamps are open



To **permanently** lock flow rate, remove Lock Guard and press red ring down. There will be a tactile and audible response. The regulator can not be unlocked.

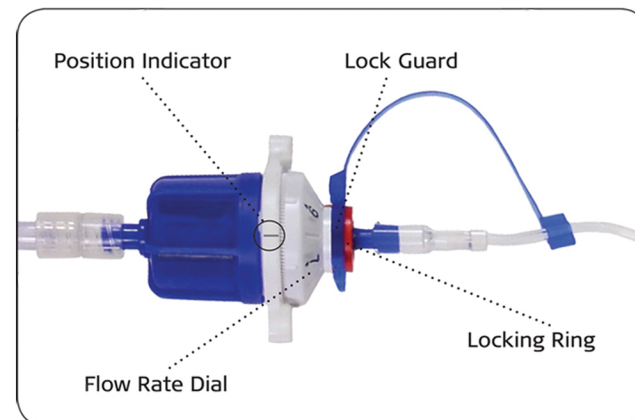

For Questions regarding the Ambu ACTION[™] Pain Pump please call our 24 hour clinical Hotline: (800) 395-2890.