

# Ambu® Aura™ Family

Quality that runs in the family



# Ambu



# THE AURA FAMILY

## Tried. Tested. Trusted.

### The Ambu legacy

Since 1937, Ambu has been rethinking medical solutions to save lives and improve patient care. The manifestations of our efforts range from early inventions like the Ambu Bag and the legendary BlueSensor™ electrodes to the world's first single-use flexible video scope – the aScope™.

### The Ambu Aura

For nearly two decades, Ambu has specialized in developing supraglottic airways. Molded in a single piece and featuring a novel anatomical curve, the AuraOnce™ is the original anatomically-curved SGA and has delivered fast and easy insertion in millions of cases. Aura-i™ is designed for everyday use, but features a built-in backup plan if needed; intubation capability with your standard ET tubes.

Our most recent innovation – AuraGain™ – integrates gastric access and intubation capability in an anatomically-curved single-use laryngeal mask.

Over time, the Aura family has expanded to cover a comprehensive range of high-quality SGAs for your needs. All members of the Aura family are made from soft materials that perform in routine as well as advanced cases.

Every single one of our masks is designed with patient safety and user friendliness in mind, and our solutions are known for adding new value to our customers. We call it the Ambu Aura.

**Ambu**



## Ambu AuraGain

The AuraGain is Ambu's 2nd-generation laryngeal mask, satisfying three fundamental airway management needs by integrating gastric access and intubation capability into the original anatomically-curved single-use device that facilitates rapid establishment of a safe airway.

### Additional features include:

- Thin and soft cuff designed to deliver high seal pressures (documented up to 40cm H<sub>2</sub>O\*) while minimizing patient airway trauma
- Patented stabilizer pad to help maintain mask placement and seal during patient movement or transport
- Patented dead space reduction feature for pediatric patients
- Integrated bite block



\*data on file

### Technical Specifications and ordering information

| Item no.  | Size | Patient weight | Max. inflation volume cuff | Max. intra-cuff pressure | Max. gastric tube | Max. ETT tube | Connector | Material           |
|-----------|------|----------------|----------------------------|--------------------------|-------------------|---------------|-----------|--------------------|
| 408100000 | 1    | <5 kg          | 4 ml                       | 60 cm H <sub>2</sub> O   | 6 FR              | 3.5           | 15 mm     | Phthalate-free PVC |
| 408150000 | 1 ½  | 5-10 kg        | 7 ml                       | 60 cm H <sub>2</sub> O   | 8 FR              | 4.0           | 15 mm     | Phthalate-free PVC |
| 408200000 | 2    | 10-20 kg       | 10 ml                      | 60 cm H <sub>2</sub> O   | 10 FR             | 5.0           | 15 mm     | Phthalate-free PVC |
| 408250000 | 2 ½  | 20-30 kg       | 14 ml                      | 60 cm H <sub>2</sub> O   | 10 FR             | 5.5           | 15 mm     | Phthalate-free PVC |
| 408300000 | 3    | 30-50 kg       | 20 ml                      | 60 cm H <sub>2</sub> O   | 16 FR             | 6.5           | 15 mm     | Phthalate-free PVC |
| 408400000 | 4    | 50-70 kg       | 30 ml                      | 60 cm H <sub>2</sub> O   | 16 FR             | 7.5           | 15 mm     | Phthalate-free PVC |
| 408500000 | 5    | 70-100 kg      | 40 ml                      | 60 cm H <sub>2</sub> O   | 16 FR             | 8.0           | 15 mm     | Phthalate-free PVC |
| 408600000 | 6    | >100 kg        | 50 ml                      | 60 cm H <sub>2</sub> O   | 16 FR             | 8.0           | 15 mm     | Phthalate-free PVC |



Now featuring an enhanced cuff to support efficient & secure mask placement

## Ambu AuraOnce

The AuraOnce was the first laryngeal mask offered by Ambu and remains the ideal airway solution for routine cases. Featuring the original anatomical curve, the AuraOnce was designed for rapid insertion with the goal of minimizing pressure on the airway as it naturally conforms to the shape of the human anatomy.




### Technical Specifications and ordering information

| Item no.   | Size | Patient weight | Max. inflation volume cuff | Max. intra-cuff pressure | Max. ETT tube | Connector | Material           |
|------------|------|----------------|----------------------------|--------------------------|---------------|-----------|--------------------|
| 321100000U | 1    | <5 kg          | 4 ml                       | 60 cm H <sub>2</sub> O   | 3.5           | 15 mm     | Phthalate-free PVC |
| 321150000U | 1 ½  | 5-10 kg        | 7 ml                       | 60 cm H <sub>2</sub> O   | 4.0           | 15 mm     | Phthalate-free PVC |
| 321200000U | 2    | 10-20 kg       | 10 ml                      | 60 cm H <sub>2</sub> O   | 5.0           | 15 mm     | Phthalate-free PVC |
| 321250000U | 2 ½  | 20-30 kg       | 14 ml                      | 60 cm H <sub>2</sub> O   | 5.5           | 15 mm     | Phthalate-free PVC |
| 321300000U | 3    | 30-50 kg       | 20 ml                      | 60 cm H <sub>2</sub> O   | 6.5           | 15 mm     | Phthalate-free PVC |
| 321400000U | 4    | 50-70 kg       | 30 ml                      | 60 cm H <sub>2</sub> O   | 7.5           | 15 mm     | Phthalate-free PVC |
| 321500000U | 5    | 70-100 kg      | 40 ml                      | 60 cm H <sub>2</sub> O   | 8.0           | 15 mm     | Phthalate-free PVC |
| 321600000U | 6    | >100 kg        | 60 ml                      | 60 cm H <sub>2</sub> O   | 8.0           | 15 mm     | Phthalate-free PVC |

Now featuring an enhanced cuff to support efficient & secure mask placement

## Ambu Aura-i

Ambu Aura-i is a disposable laryngeal mask which can be utilized as a conduit for scope-guided intubation. Anatomically curved to match the natural shape of the human airway, the Aura-i provides fast and easy placement and long-term performance.

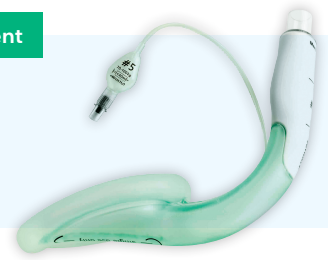


### Technical Specifications and ordering information

| Item no.   | Size | Patient weight | Max. inflation volume cuff | Max. intra-cuff pressure | Max. ETT tube | Connector | Material           |
|------------|------|----------------|----------------------------|--------------------------|---------------|-----------|--------------------|
| 329100000U | 1    | <5 kg          | 4 ml                       | 60 cm H <sub>2</sub> O   | 3.5           | 15 mm     | Phthalate-free PVC |
| 329150000U | 1 ½  | 5-10 kg        | 7 ml                       | 60 cm H <sub>2</sub> O   | 4.0           | 15 mm     | Phthalate-free PVC |
| 329200000U | 2    | 10-20 kg       | 10 ml                      | 60 cm H <sub>2</sub> O   | 5.0           | 15 mm     | Phthalate-free PVC |
| 329250000U | 2 ½  | 20-30 kg       | 14 ml                      | 60 cm H <sub>2</sub> O   | 5.5           | 15 mm     | Phthalate-free PVC |
| 329300000U | 3    | 30-50 kg       | 20 ml                      | 60 cm H <sub>2</sub> O   | 6.5           | 15 mm     | Phthalate-free PVC |
| 329400000U | 4    | 50-70 kg       | 30 ml                      | 60 cm H <sub>2</sub> O   | 7.5           | 15 mm     | Phthalate-free PVC |
| 329500000U | 5    | 70-100 kg      | 40 ml                      | 60 cm H <sub>2</sub> O   | 8.0           | 15 mm     | Phthalate-free PVC |
| 329600000U | 6    | >100 kg        | 60 ml                      | 60 cm H <sub>2</sub> O   | 8.0           | 15 mm     | Phthalate-free PVC |

Now featuring an enhanced cuff to support efficient & secure mask placement

## Ambu AuraStraight™

Ambu AuraStraight is an alternative to the popular AuraOnce anatomically-curved laryngeal mask, offering users a choice for those who prefer a straighter airway tube for their routine cases.

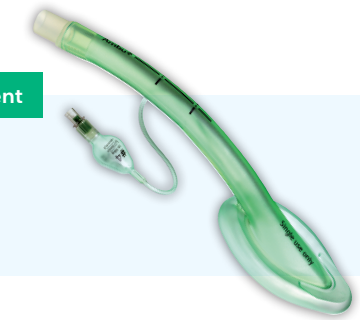

### Technical Specifications and ordering information

| Item no.   | Size | Patient weight | Max. inflation volume cuff | Max. intra-cuff pressure | Connector | Material           |
|------------|------|----------------|----------------------------|--------------------------|-----------|--------------------|
| 324100000U | 1    | <5 kg          | 4 ml                       | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 324150000U | 1 ½  | 5-10 kg        | 7 ml                       | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 324200000U | 2    | 10-20 kg       | 10 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 324250000U | 2 ½  | 20-30 kg       | 14 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 324300000U | 3    | 30-50 kg       | 20 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 324400000U | 4    | 50-70 kg       | 30 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 324500000U | 5    | 70-100 kg      | 40 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 324600000U | 6    | >100 kg        | 60 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |

Now featuring an enhanced cuff to support efficient & secure mask placement

## Ambu AuraFlex™

Ambu AuraFlex is a flexible laryngeal mask which is specially designed for ENT, ophthalmic, dental and other head and neck surgeries.

Its wire-reinforced, kink-free airway tube allows for optimal positioning away from the surgical field without loss of seal or risk of occlusion.



### Technical Specifications and ordering information

| Item no.   | Size | Patient weight | Max. inflation volume cuff | Max. intra-cuff pressure | Connector | Material           |
|------------|------|----------------|----------------------------|--------------------------|-----------|--------------------|
| 327200000U | 2    | 10-20 kg       | 10 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 327250000U | 2 ½  | 20-30 kg       | 14 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 327300000U | 3    | 30-50 kg       | 20 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 327400000U | 4    | 50-70 kg       | 30 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 327500000U | 5    | 70-100 kg      | 40 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |
| 327600000U | 6    | >100 kg        | 60 ml                      | 60 cm H <sub>2</sub> O   | 15 mm     | Phthalate-free PVC |



# Our Packaging is Shrinking



When it comes to sustainability, Ambu aims to lead by example. That's why we have adopted a circular approach in every aspect of our products' journey, not only for the benefit of healthcare professionals and patients, but also for our planet.

One example of that is a new packaging design we have implemented for our laryngeal masks that lowers our carbon footprint. We have redesigned the packaging material with significantly less plastic, and we are utilizing bioplastics in the raw material we use to produce the cuff protectors.

**As we take leaps toward a more sustainable future, we hope to inspire others – after all, the more who join, the greater the impact.**



Ambu, Inc.  
6721 Columbia Gateway Drive  
Suite 200  
Columbia, MD 21046  
Tel. 800 262 8462  
Fax 800 262 8673  
[ambuUSA.com](http://ambuUSA.com)



Questions about Ambu laryngeal masks? Contact your Ambu representative at **800-262-8462** or visit [ambuUSA.com/laryngeal-masks](http://ambuUSA.com/laryngeal-masks)

**Scan QR code** to discover more about Ambu's sustainability efforts.