


INSTRUCTIONS FOR USE

Ambu® King Mask

Ambu

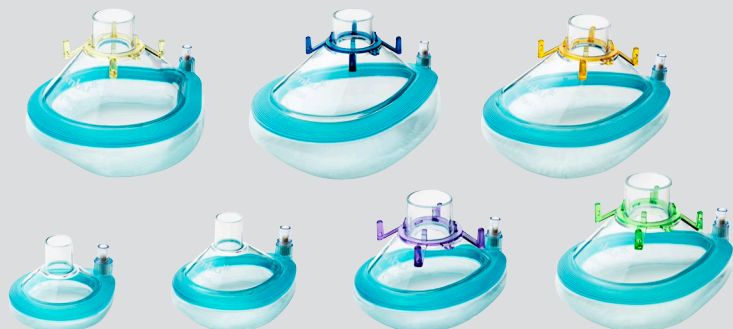

Intended use

Ambu King Masks are intended to direct anesthesia or medical gases into the patient's upper airway without introducing any apparatus into the trachea.

The device is for use in hospital and pre-hospital environment.

For use by CPR trained personnel only.

Disposable Anesthesia Face Mask.

Non-sterile, Non-conductive, with check valve.

Single Patient Use Face Mask.

Not made with natural rubber latex.

Phthalate free.

This product is MR Safe.

Instructions for use


- Remove the mask from the package and inspect the device. In case of damage, dirt or deformation discard the mask and have a replacement mask available.
- Check the cushion's inflation for appropriate fit and seal to the patient's face.
- The mask collar, if present, may be removed if not needed.
- To adjust cushion inflation, insert a standard luer tip syringe into the inflation valve. Inflate or deflate as needed.
- If deemed necessary, the surface of the product can be manually cleaned with water and a mild detergent. Rinse thoroughly with clean water. Allow to dry, and check that the mask is fully operational before re-use on the same patient. If not fully operational, the mask shall be discarded.
- Discard after single patient use according to hospital protocol.

WARNINGS

- Avoid contact with sharp objects which could damage cuff.
- For single patient use only.
- Use on other patients can cause cross infection.
- The design and material of the device are not compatible with automatic cleaning or sterilization processes, as it may leave harmful residues or cause malfunction of the device.
- Scented masks are contraindicated for patients with known allergies to perfumes and plastics.

CAUTIONS

- Over-inflation or excessive pressure to hold the mask in place may restrict or close the patient's airway.
- Under-inflation may result in poor seal and leakage.
- Mask is designed to connect to a standard fitting. The inside diameter of the mask port of #1-2 is non-standard and must not be attached to other medical device. The outside diameter of the mask port of #3-7 is non-standard and must not be attached to other medical device.
- US federal law restricts this device to sale by or on the order of a licensed health care practitioner.

Symbol indications	Description
	MR Safe

Storage temperature range: -40 °C – +60 °C
Operation temperature range: -18 °C – +50 °C

Connector size:

Ambu King Mask sizes: #1 – #2, 15 mm OD connector
Ambu King Mask sizes: #3 – #7, 22 mm ID connector

Ambu® is a registered trademark of Ambu A/S, Denmark
Ambu A/S is certified according to ISO 13485

Ambu



Ambu A/S

Båtorpbakken 13

2750 Ballerup

Denmark

T +45 72 25 20 00

ambu.com

US: Rx only