

QUICK REFERENCE GUIDE

Ambu® aScope™ 4 Cysto

Insertion portion

| | |
|----------------------------------|--------------------|
| Bending angle | 210° up, 120° down |
| Working length | 390 mm (15.4") |
| Insertion cord outer diameter | 16.2 Fr / 5.4 mm |
| Minimum instrument channel width | 6.6 Fr / 2.2 mm |



Image Extend

To extend the live image, double click on the live image.
To remove "Image Extend," double click on the image again.

Insertion portion


The aScope 4 Cysto has been designed to be used with:

- Irrigation set (line and sterile water or saline bag) with Luer connection
- Syringe and other Luer connecting accessories
- Endoscopic accessories labeled for use in a minimum working channel size of (ID) 2.0 mm / 6.0 Fr or less*
- Holmium YAG laser (2.1 microns wavelength)
- High-frequency surgical equipment fulfilling EN 60601-2-2.0 to keep high-frequency leakage currents within allowed limits, the maximum sinus peak voltage level of the electrosurgical unit shall not exceed 2.2 kVp



*There is no guarantee that instruments selected solely using this minimum working channel size will be compatible in combination. Compatibility of selected instruments should be tested before the procedure.

Lubrication

If the camera image of the aScope 4 Cysto becomes unclear, the distal end can be cleaned by gently rubbing it against the mucosal wall, or withdraw the cystoscope and clean the distal end.

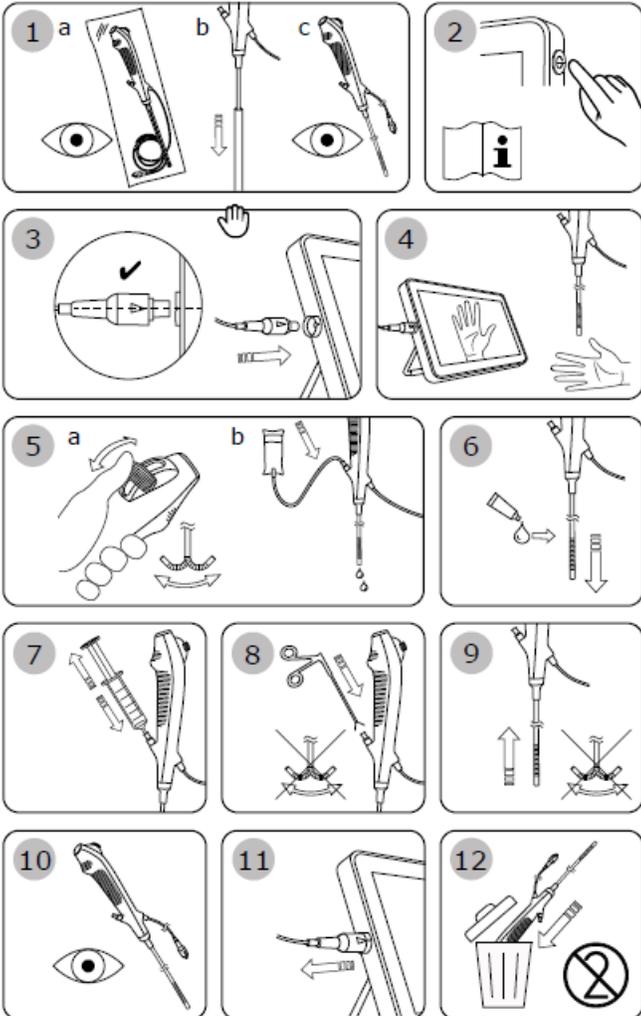

Deflection

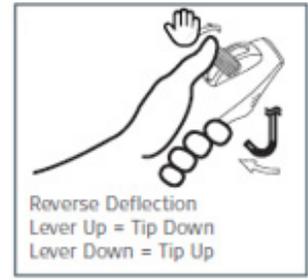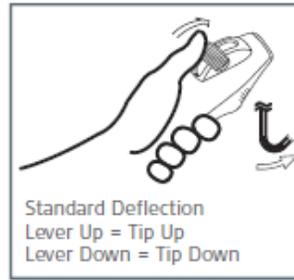

601001000 Canada Only
601101000 US Only

600001000 Canada Only
600101000 US Only

Accessories



Accessories

Stopcock 4-way 50psi (3.5 bar). Swivel Nut, Large Bore

K12-11412

50 pcs

Hemostasis Valve. AccessPLUSTM, Large Bore

K12-11413

25 pcs

To download Instructions For Use for the aScope 4 Cysto, scan the QR code below.

Ambu FOREVER FORWARD

Ambu, Inc.
6721 Columbia Gateway Drive, Suite 200
Columbia, Maryland 21046
Tel. 800 262 8462
Fax 800 262 8673
ambuUSA.com

CE 2797



US: Rx only