
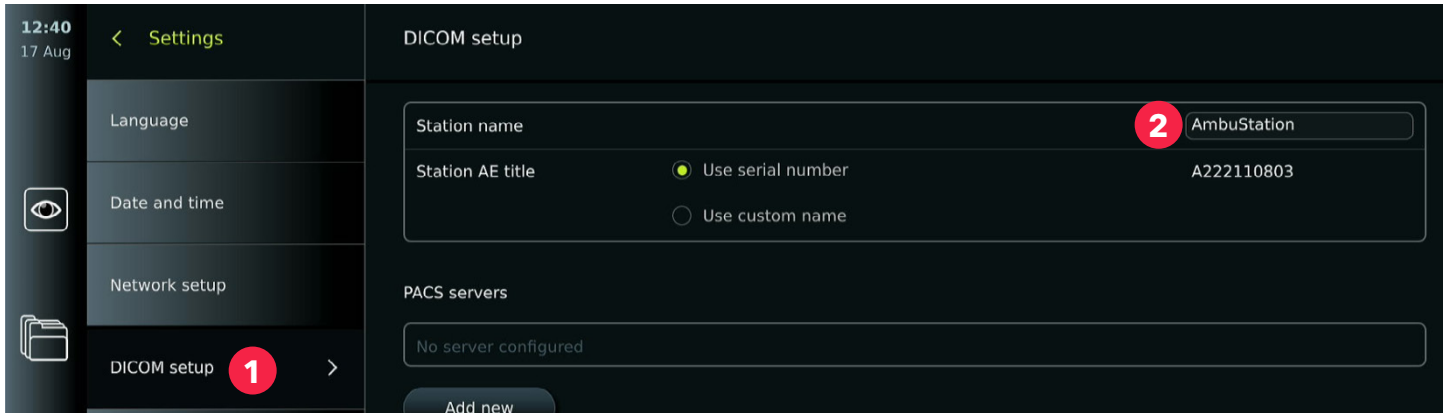


PACS Connectivity and Worklist Set Up

Quick Start Guide


1. PACS SET UP

1. To set up the connection to your PACS server, first ensure you are logged in as Administrator and connect to WiFi.
2. On the left hand side of the screen, navigate to **Settings**  > **Setup** > **DICOM Setup** **1**.
3. Enter the device name in the **Station name** **2** field. This is a free-text field and should be however you would like the device to be identified by the PACS server.




2. CONNECT TO THE PACS SERVER

4. In the DICOM setup screen, under **PACS servers**, press **Add new** **3**.

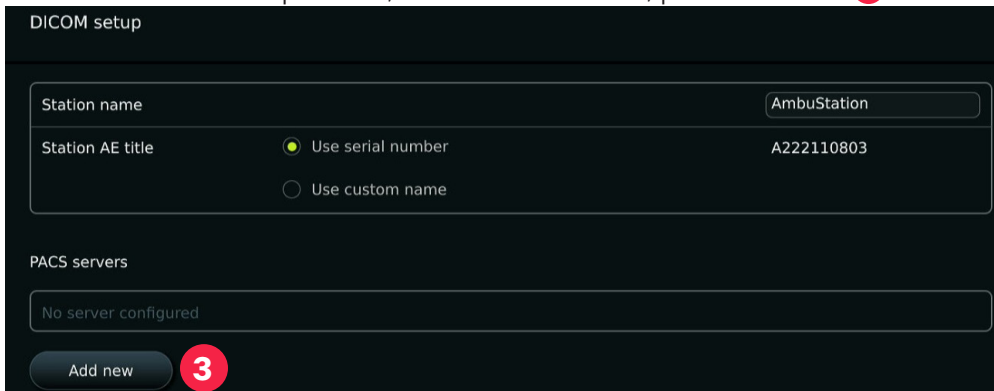

5. Enter a name for the PACS connection in the **PACS name** **4** field.
6. Enter the relevant details in the **PACS AE title**, **Host name** and **Port number** **5** fields and select the required **TLS** **6** setting.
7. Verify the connection by pressing **Test connection** **7**.
8. If successful, a confirmation message will appear. Then press **Create** **8** to save the server.

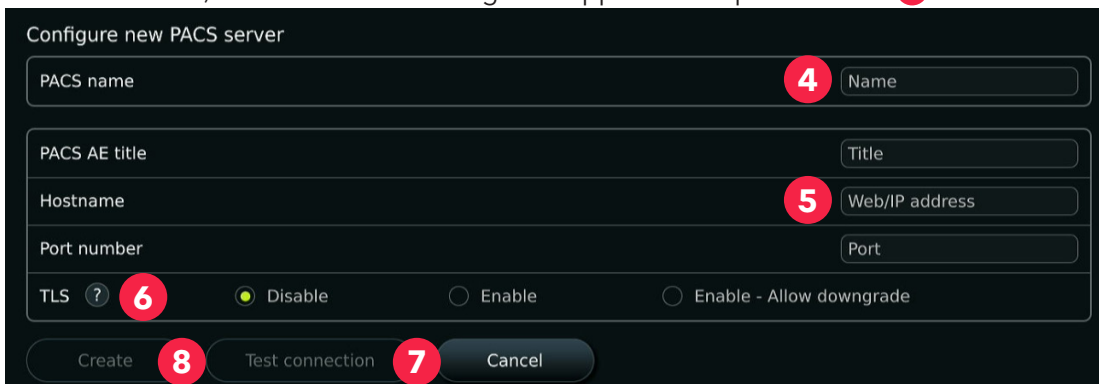

3. WORKLIST SET UP

CONNECT TO THE WORKLIST



9. Within the DICOM setup screen, under **Worklist settings**, switch the slider to green to **Enable Worklist** 9 .
10. Enter the relevant details in the **Worklist server AE title**, **Worklist server hostname** and **Worklist server port number** 10 fields and select the required **TLS** 11 and **Modalities** settings.
11. Input the relevant number of hours in the **Display/Hide procedures** 12 fields. These will determine the time period from the worklist that the customer information will be taken for.
12. Verify the connection by pressing **Test worklist connection** 13 . If successful, a confirmation message will appear.

The screenshot shows the 'DICOM settings' screen with the 'Worklist settings' section expanded. The 'Enable worklist' toggle is turned on (green). The 'Worklist server AE title' is 'FLUX_WORKLIST'. The 'Worklist server hostname' is 'worklist.fluxinc.ca'. The 'Worklist server port number' is '1070'. The 'TLS' setting is 'Disable'. The 'Modalities' setting is 'MR'. The 'Display upcoming procedures (hours)' and 'Hide past procedures older than (hours)' are both set to '24'. A 'Test worklist connection' button is at the bottom.



4. SELECTING PATIENTS FROM A WORKLIST


In Live view , you should now see the worklist at the top-left of the screen

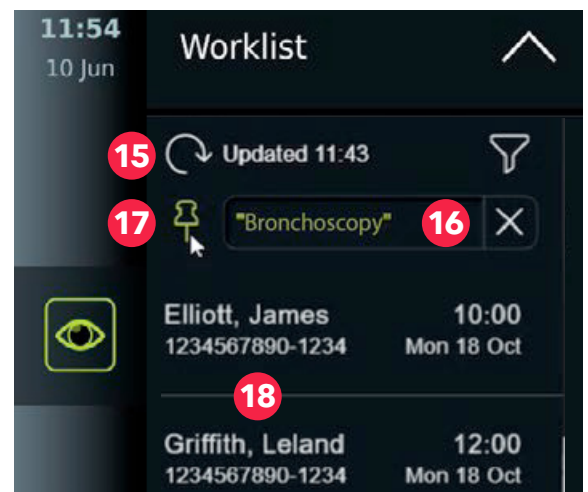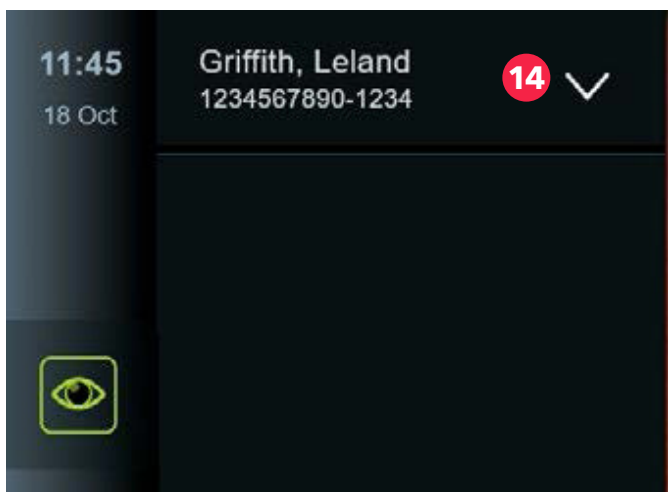13. Press the **arrow** 14 icon open the patient list.
14. Press the **update** 15 icon to retrieve patient information.
15. Press the **search** 16 field to search by a search term.
Press the **pin** 17 icon to keep the search term active and then deactivate the search term.
16. Select a **patient** 18 from the list and then press **Confirm** on the pop-up.
To select a different patient, select the other patient and press **Change**.
To deselect the patient, select the patient's name and press **Deselect**.



5. EXPORTING TO PACS

- At the beginning of the procedure, once your endoscope is inserted, select a patient from the worklist dropdown. Then during the procedure, use the capture buttons to take images as needed.
- Once the procedure is completed and you remove the scope, the system will ask you to finish and export. This will save the files within the file management  on the monitor. To export, go into the procedure folder, and select the files you wish to export by tapping the check box at the bottom right hand corner of the image.
- With your images selected, in the top right hand corner of the screen select the Export Icon . Then on the left hand side, select **DICOM** **19** under **Export format**.
- Patient information will be filled out. Or if patient information wasn't retrieved from a worklist, it can be entered manually **20**.
- Select the required **PACS server** **21** under **PACS**.
- Press **Export** **22**.
- Successful transfer will be confirmed via a pop-up.

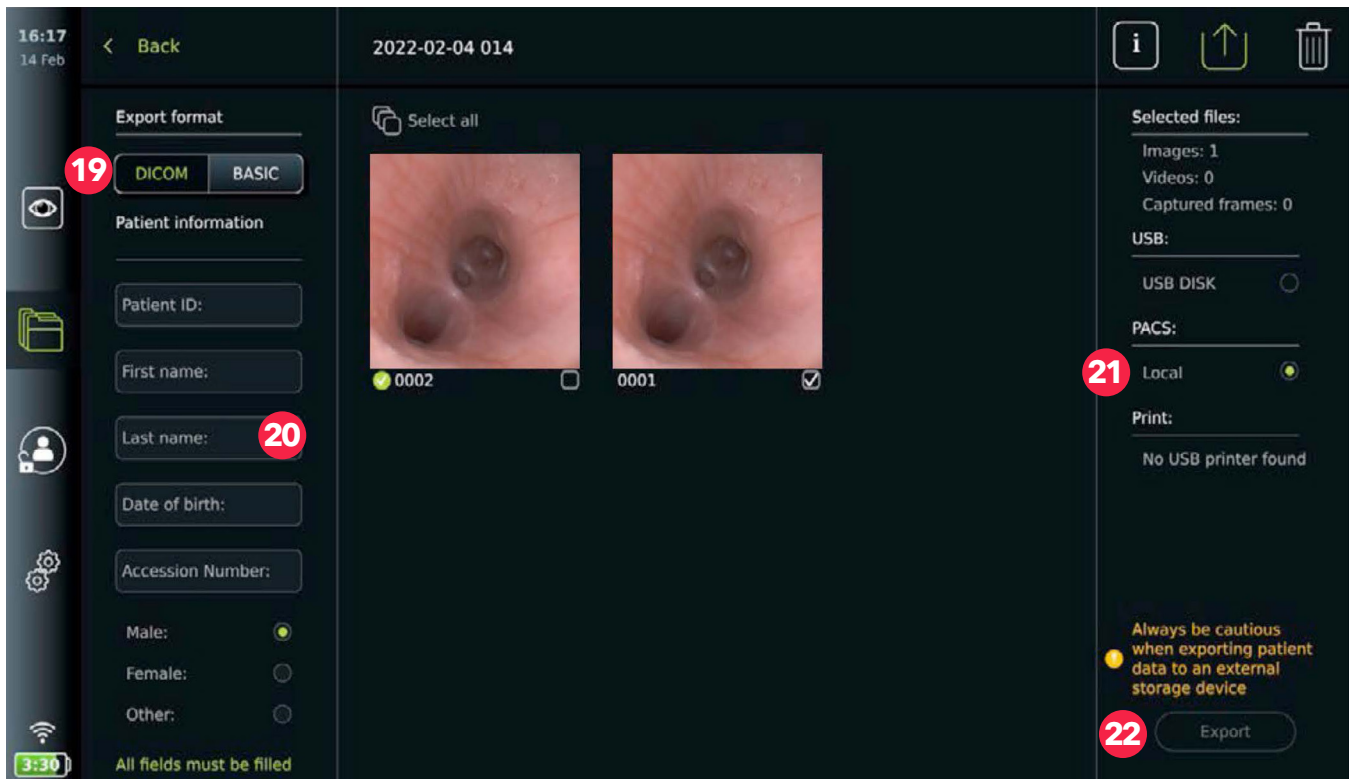

Ambu FOREVER FORWARD

Ambu, Inc.
6721 Columbia Gateway Drive, Suite 200
Columbia, Maryland 21046
Tel. 800 262 8462
Fax 800 262 8673
ambuUSA.com



Refer to IFU for
full information